



NuEra

FUTURE PERFECT

NuEra™ tight

High powered next generation
temperature control.

Non-invasive cellulite reduction system.

All over perfect with natural looking results.



LUMENIS.COM/AESTHETIC

Azani
Medical Spa

Lumenis®
Energy to Healthcare

Towards Future Perfect

NuEra *tight* uses APIC (Automatic Power Impedance Control), the next generation temperature control technology, to ensure safety and efficacy.

APIC automatically increases the RF power to the target temperature very gradually and then maintains it, delivering precise heating to the skin's surface with predictable and consistent results.

This leaves you to focus on treating your patients with the light and ergonomic handpiece while letting the system take care of delivering the right temperature.

NuEra Tight is **Future Perfect**.



A new future for patients. And for you.

Cellulite is a problem that affects everyone at some point, but now there's a new era that provides fast, effective treatment: NuEra *tight*.


Introduced by Lumenis®, pioneers in energy-based technologies, NuEra *tight* is FDA-cleared for RF heating to reduce cellulite temporarily. For patients, NuEra *tight* means new look and comfort. For you, NuEra *tight* means simplicity and speed.

Look forward to a whole new future for your business.

Top-to-toe: targeted skin smoothing

NuEra *tight* is an RF solution for skin laxity in the most problematic areas: around eyes, mouth, cheeks and neck and on inner and outer thighs, upper arms, buttocks and abdomen.

Through powerful RF waves emitted at 470kHz and high 250W power, NuEra *tight* delivers the required amount of heat - precisely and deeply - for optimally effective treatment and consistently successful results.

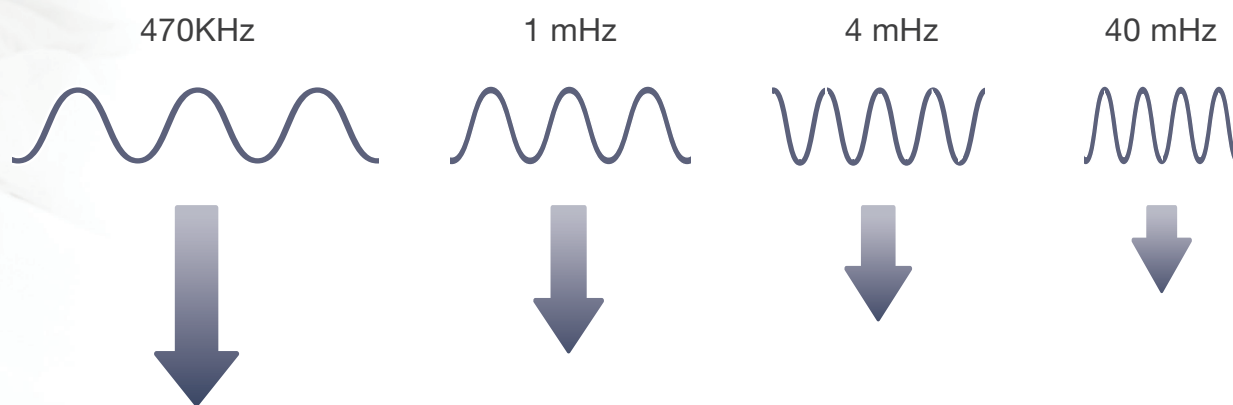


The lower the RF frequency, the deeper the skin tissue penetration and the better the thermal effect and results. NuEra *tight* RF emits at 470KHz to infiltrate deep down.

Power perfect: temperature-controlled RF

NuEra *tight* uses APIC (Automatic Power Impedance Control), the next generation temperature control technology, to ensure safety and efficacy.


APIC automatically increases the RF power to the target temperature very gradually and then maintains it, delivering precise heating to the skin's surface with predictable and consistent results.

This leaves you to focus on treating your patients with the light and ergonomic handpiece while letting the system take care of delivering the right temperature.

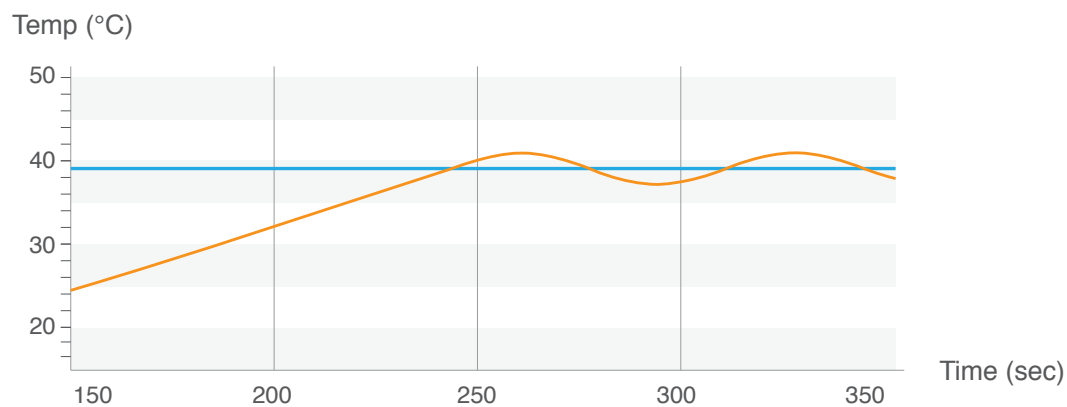

APIC monitors skin tissue temperature and impedance in real time. It automatically adjusts the RF power for heat to penetrate deeper and more uniformly.

Powerful.
Versatile.
Simple.

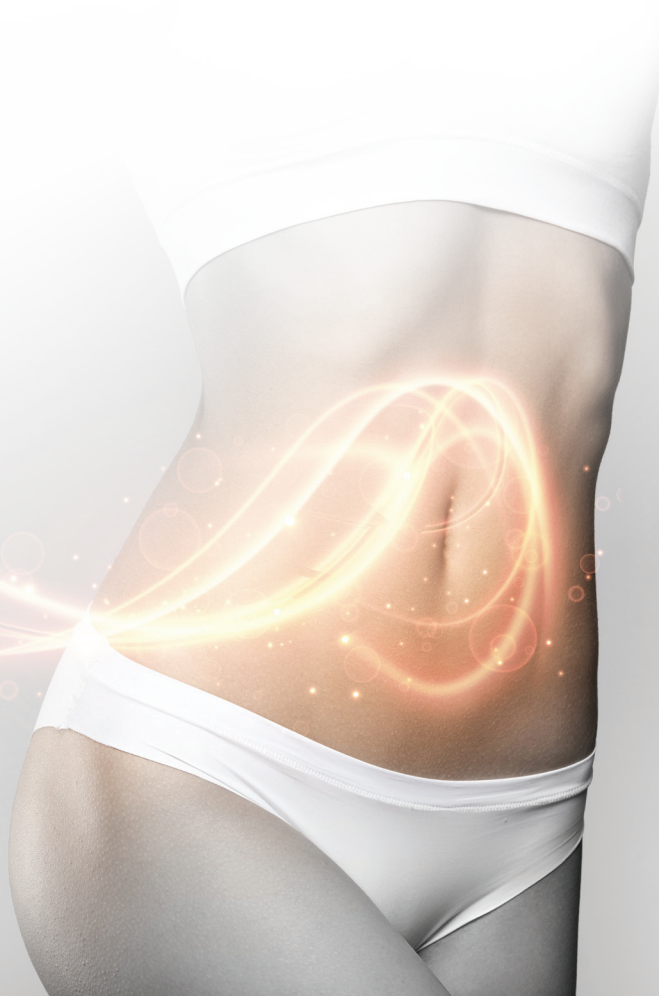

Precision perfect: how NuEra tight works


NuEra *tight* is user-friendly: it features an automatic configuration graphic interface for maximum treatment versatility.

Simply select the area of the body or face to treat and the system automatically sets the correct parameter and displays the right accessory and treatment protocol.

The NuEra *tight* electrode is applied to the skin's surface and the device is set to a specific target temperature. The electrode is gently moved in a sweeping or circular fashion while gradually heating up the skin surface, providing a safe, pleasant and comfort treatment.

As the skin's temperature rises to the target temperature, the RF output automatically adjusts for precise heating due to the APIC technology. A targeted zone is treated for approximately 8-12 minutes to achieve the clinical goal.

Select treatment area

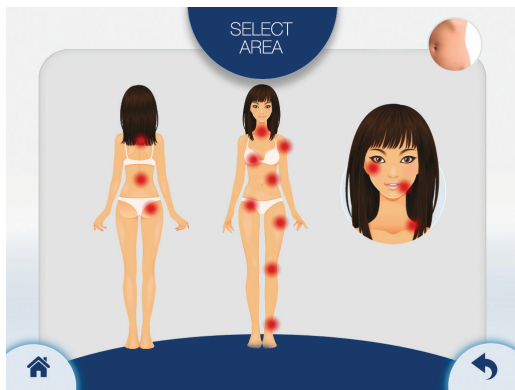

Recommended treatment



Success for face, neck and body

Thanks to its remarkable 250W power, NuEra *tight* uniquely works with one resistive bipolar handpiece and two unipolar handpieces with Coated Capacitive Energy Transfer (CCET) electrodes in a wide variety of capacitor head sizes - from small to very large.

The system treats all areas quickly, safely and efficiently in a warmly soothing experience that your patients will enjoy.

In just a few short treatment sessions, patients can look forward to a new self with no downtime.



NuEra tight is designed with a unique feature set and user-friendly graphic interface, which enables maximum versatility for top to toe treatments for safe, warmly soothing experience.

Impressive results (Before & After)



Before



After



Before



After



Your journey with NuEra tight has just begun.

We want your business to grow with NuEra *tight*. That's why we have put together a support package that begins with the first conversation and will stay with you as your business begins to see the benefits. We provide a support package which ensures that you have all the training you need and receive the most up to date clinical materials.



Lumenis Practice Locator

Thousands of patients search for treatment providers every month. Help them find your clinic by registering your practice on the Lumenis Practice Locator. We run campaigns through influencers, social media and viral videos to promote this directory.



Lumenis.com

Consumers have many questions about aesthetic and women's health treatments. Our dedicated patient-facing website will give them the re-assurance and confidence they need to move forward with the treatments.



Team Training

We install the system and train your staff, so that we are able to assess skin types and conditions, creating the perfect treatment plan for each individual patient. Our expert clinical training means they will handle consultations and treatments with confidence.



Your Most Valuable Resource

Your dedicated representative is an email, text, or phone call away. We want you to feel part of the Lumenis family, so please feel free to contact us any time. We'll always do our best to get back to you within 24 hours.



Brand Your Clinic

As a Lumenis client you will have access to the Lumenis Physician Portal. Here, you can find materials such as brochures and posters. As well as gain access to a wealth of clinical and product information.



Best Practice Marketing

Your individual Lumenis representative will be able to share marketing plans which you can tailor to your own business. These range from 90 day benchmarks for success to loyalty programs and promotions.

Specifications

Emission Frequency	470 kHz \pm 10%
Output Energy	250 W Max
Display LCD	Touch Screen 10.4"
Coupling System	Capacitive
Operative Mode	Automatic & Manual
Impedance measurement	Automatic
Temperature Control	Automatic
Temperature Adjustment	37° \div 45° \pm 1°C
Timer	Max 60 min adj.
Feeding	110 VAC - 60 Hz
Dimensions with Stand	68x61x117 cm (WxDxH)
Weight	15Kg
Safety Class	Class1 - Type BF
Standard accessories	
Handpieces	3: face and body, bipolar
Return Electrodes	2: 1 handpiece and 1 plates
Isolated electrodes with temperature sensor (CET)	7: \varnothing 20, \varnothing 30, \varnothing 40, \varnothing 60, \varnothing 70, \varnothing 80, \varnothing 100 mm
Cosmetics	NuEra Face cream, NuEra Body cream

About Lumenis

Lumenis is a global leader in the field of minimally invasive clinical solutions for the surgical, ophthalmology and aesthetic markets, and is a world-renowned expert in developing and selling energy-based technologies, including laser, intense pulsed light (IPL) and radio frequency (RF). For nearly 50 years, Lumenis' ground-breaking products have redefined medical treatments and have set numerous technological and clinical gold standards. Lumenis has successfully created solutions for previously untreatable conditions, as well as designed advanced technologies that have revolutionised existing treatment methods in each and every one of the verticals we operate in. Our drive for innovation stems from an uncompromising commitment to improving the health and well-being of our patients; addressing new and growing needs of aging populations; and offering medical professionals cutting-edge solutions that meet the demands of modern life. The world over, we bring Energy to Healthcare.

Distributor of Lumenis
"Concern - Energomash" CJSC
Azatutyan avenue 26/8, Yerevan 0014, Armenia
Tel.: 20 97 77, 20 97 75, 20 97 74
E-mail: info@c-e.am
www.bellice.am, www.c-e.am

